


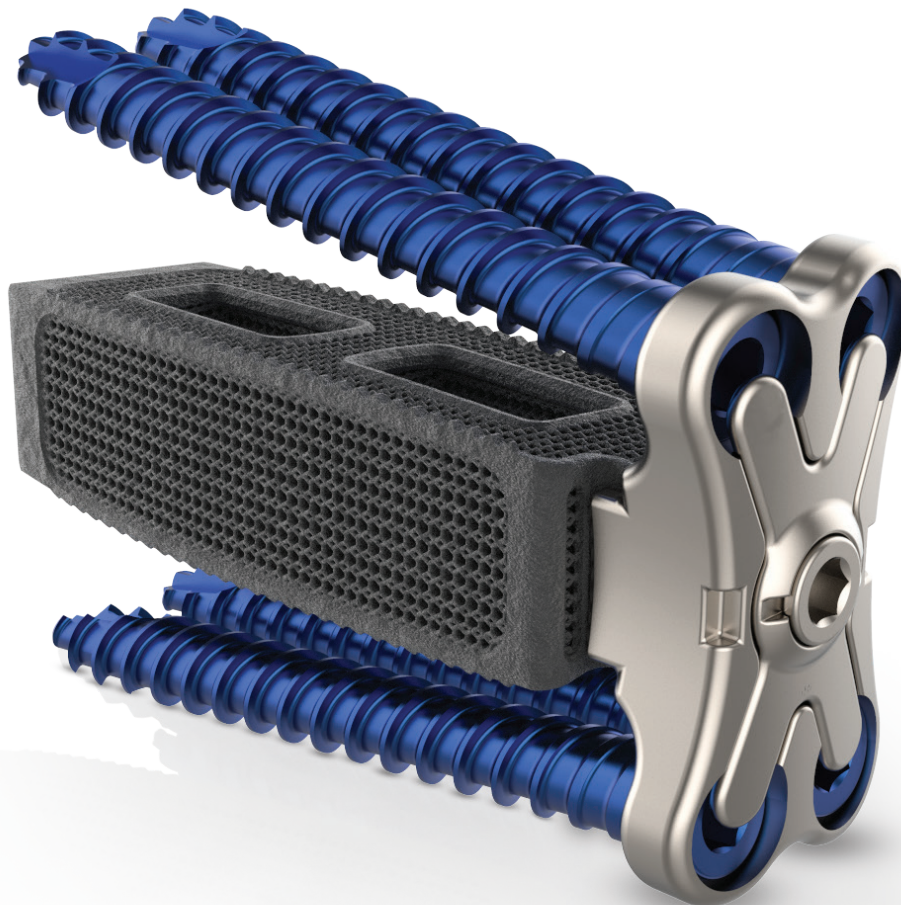

A New Foundation
For Growth

TrellOss[®] -L MPF

Porous Ti Interbody System



 **ZimVie**



A New Foundation for Growth

Introducing TrellOss - L MPF Porous Ti Interbody System

A 3D printed titanium interbody platform featuring a scaffold structure with 70% porosity and a 7 micron roughened surface topography to foster a cellular relevant environment for adhesion and bone ingrowth.¹

Porosity


Open architecture with 70% porosity including varying pore sizes of 300, 500, and 700 microns that mimic cancellous bone allowing for a conducive environment for cellular activity^{1,5,6,7}

Structure


Scaffolding structure provides additional surface area ^{2,3} and an elastic modulus similar to PEEK⁸

Texture


7 micron surface texturing enhances the wicking nature⁹ and creates an environment for potential cellular adhesion^{2,3,4}

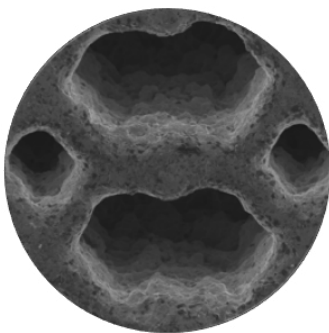


SEM image of TrellOss Surface at **50x** magnification

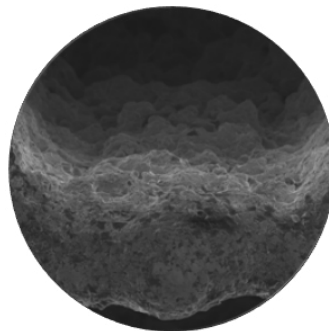


SEM image of TrellOss Surface at **100x** magnification

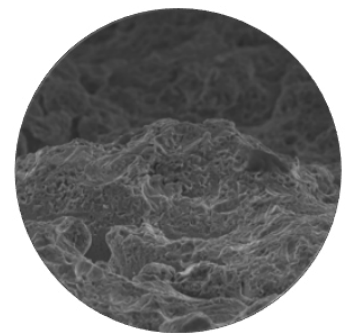

SEM image of TrellOss Surface at **450x** magnification

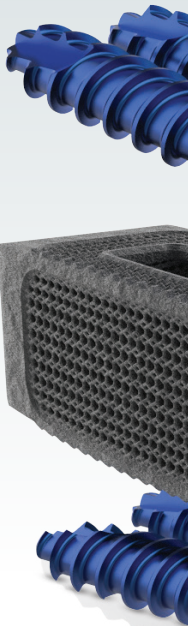

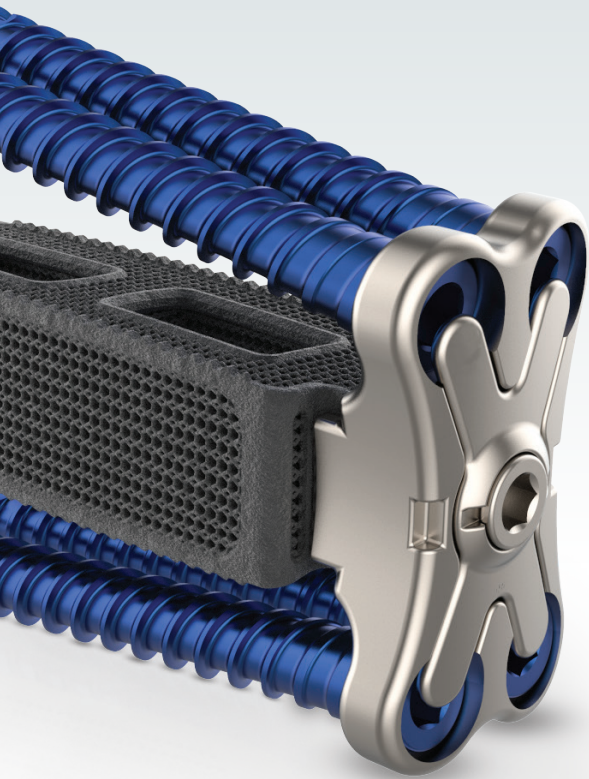

TrellOss-L Implant


- Rigid teeth help to resist implant migration
- Bullet-tip nose to aid in implant insertion
- Central windows for graft packing and containment
- Implants are sterile-packed to reduce the risk of contamination and hospital reprocessing costs
- Compatible with all MPF plates

TrellOss-L Sizes


Widths	Lordosis	Heights**	Lengths***
18 mm 22 mm	0°, 8°	8 mm–12 mm	45 mm–60 mm
18 mm 22 mm	0°, 8°	14 mm	50 mm–60 mm
18 mm 22 mm	14°	10 mm–14 mm	45 mm–60 mm
18 mm 22 mm	14°	16 mm	50 mm–60 mm
22 mm	20°	12 mm–18 mm	45 mm–60 mm

MPF Sizes

Plates	Heights**	Lengths
1-hole	8 mm–16 mm	10.5 mm–14.5 mm*
2-hole	8 mm–14 mm	21 mm–27 mm***
4-hole	6 mm–14 mm	24.5 mm–32.5 mm***

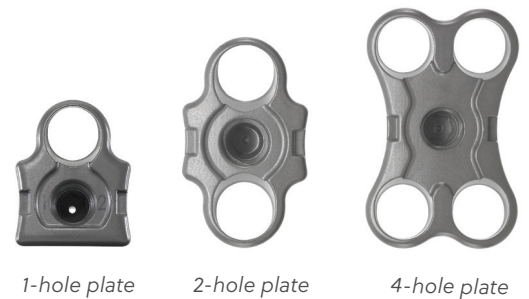

Screws	Diameters	Lengths***
Self-tapping	5.5 mm 6.0 mm	30 mm–60 mm
Bi-cortical	5.5 mm 6.0 mm	30 mm–60 mm

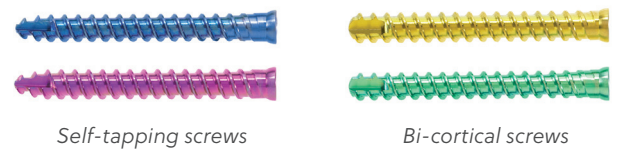

*1 mm inc **2 mm inc ***5 mm inc

References

McGilvray KC, Easley J, Seim HB, et al. Bony ingrowth potential of 3D-printed porous titanium alloy: a direct comparison of interbody cage materials in an in vivo ovine lumbar fusion model. *Spine J* 2018;18(7):1250-1260. Olivares-Navarrete R, Hyzy SL, Slosar PJ et al. Implant materials generate different peri-implant inflammatory factors: poly-ether-ether-ketone promotes fibrosis and microtextured titanium promotes osteogenic factors. *Spine* 2015;40(6):399-404. 2. Olivares-Navarrete R, Hyzy SL, Gittens RA, et al. Rough titanium alloys regulate osteoblast production of angiogenic factors. *Spine J* 2013;13(11):1563-70. 3. Rao PJ, Pelletier MH, Walsh WR, et al. Spine Interbody Implants: Material Selection and Modification, Functionalization and Bioactivation of Surfaces to Improve Osseointegration. *Orthop Surg* 2014;6:81-89. 4. Ponader S, von Wilmowsky C, Widenmayer M, et al. In vivo performance of selective electron beam-melted ti-6al-4v structures. *J Biomed Mater Res A* 2010;92A:56-62. 5. Li JP, Habibovic P, et al.: Bone ingrowth in porous titanium implants produced by 3D fiber deposition. *Biomaterials* 2007;28:2810. 6. Karageorgiou V, Kaplan D. Porosity of 3D biomaterial scaffolds and osteogenesis. *Biomaterials* 2005;26(27):5474-91. 7. Permeswaran, V., (2019) Elastic Modulus Characterization of Porous Titanium TrelOss™ Structure, 2922. 1-GLBL-en-REV1219, Zimmer Biomet Spine, Westminster, CO. 8. Permeswaran, V., (2019) Measuring the Wicking Nature of Porous Titanium TrelOss™ Structure, 2921. 1-GLBL-en-REV1219, Zimmer Biomet Spine, Westminster, CO



Manufactured by: Nexxt Spine, LLC
14425 Bergen Blvd, Suite B
Noblesville, IN 46060

ZimVie.com

 **ZimVie**
Restoring Daily Life.™

Unless otherwise indicated, all trademarks herein are the property of ZimVie Inc. and its subsidiaries, including Zimmer Biomet Spine, Inc. (collectively, "ZimVie"). Unless otherwise indicated, all products herein are manufactured, marketed, and distributed by ZimVie. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries or regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie.

ZV0893 REV A 03/23. © 2023 ZimVie Inc. All rights reserved.